INNOVA

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1. Charge method
Insert the magnetic charging cable into the power output (5V±0.25V adapter).

2. APP download and installation
1. iOS Version, search for "BitHealth HR Pro" in the App Store and download, then install, or you can scan the QR code below.
2. Android Version, search for "BitHealth HR Pro" in Google Play Store and download, or you can scan QR code.

Android

iOS

3. How to turn on device
Long press the button for 3 seconds, the device turns on and main interface will display date, time, battery and connection status.

4. Connection
1. The connection between APP and device is via Bluetooth 4.0BLE, please ensure Bluetooth is turned on before connecting.
2. Open the APP on the mobile phone, go to settings select the first item "Find Your Smart Band" to search and match the device.
3. The ID of the device will be displayed as Z7_ xxxx, which can be found in the information interface of the device.

5. Data synchronization
When the APP is turned on, the matched device will search for data sync automatically, which includes Time, Calories, Steps, Resting HR, Sleep Data, Sport Data, Historical Data, etc.

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The alarm & schedule setting requires the device to connect with the APP, please remember to choose the "Save" on the Top right corner to sync to device.

6. Call/Message Reminder Setting
Call and Message reminder can be set independently, and incoming calls can be rejected by long press.

*The Android version APP has an additional "System Notification Message Authorization", which allows the APP access to the system notification of the mobile phone. Authorization needs to be given on the device, otherwise there will no reminder displayed on the device.
*The Android users need to set the APP running in the background.

7. System Applications
After you turn on the device, it can be operated by a short/long press on the screen, or short/long press, and the operation details as below:


2. The HR/SpO2 interface, 7-10's is required to detect HR/SPO2. Please ensure that you are seated and keep the device the same height as your heart, try not to move. Short press the button you will return to the main interface screen.

3. The Blood Pressure interface, 30s will be required to detect the BP. Please ensure that you are seated and keep the device the same height as your heart, try not to move. Short press the button will return to the main interface screen.


4. The Exercise interface, long press to get in, and long press to exit. Short press the button and you will return to the main interface screen.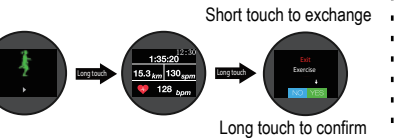


Long touch to confirm

5. The Mindfulness interface, long press to get in, and long press to exit. Short press the button and you will return to the main interface screen.

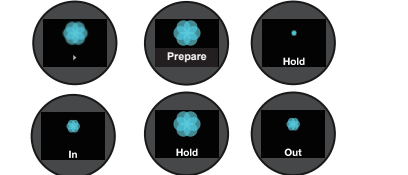
6. The Message interface, long press the screen to review latest 5 notifications, and long press the button to delete all the notifications.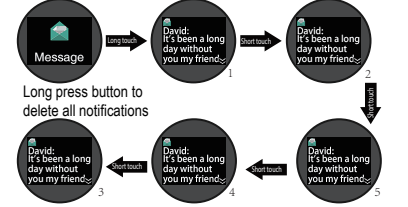

Short touch to exchange

7. The Information interface, the device ID and firmware version is displayed here, long press to power off.

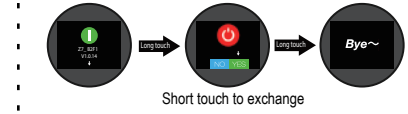Short touch to exchange

Frequently Asked Questions

1. Why doesn't the Alarm/Schedule reminder work?
Ensure that the settings have been "Saved" in App.

2. Why is there no call/message notification on Android phone?
Please ensure that you has permission to access system notifications. Make sure the APP is working in the background.

3. Why is the heart rate value not accurate?
Please ensure that the device is secured tightly on your wrist, restrict your movement and try to stay seated in one position

4. Why is there no notifications on the iPhone?
Please ensure that the Bluetooth connection is done on the APP and not on the iPhone. Otherwise you would have to re-do the process.

5. Why can't I connect automatically to Bluetooth when it's out of range?
Please re-start the device, and try to connect again.

6. Why can't I find my device after "Removing Device" in iOS APP?
To find your device please go to iPhone's Bluetooth settings, forget your device in the settings and you can find it on the APP settings.

Note: Do not expose the watch to high temperatures, the battery. It has the risk of explosion.

Declaration of conformity
Through this INNOVA CELLULAR it declares that the SPORT WATCH SUBMERSIBLE WITH HEART RATE (SW / R4M (Z4, Z7)) complies with the essential requirements and any other applicable or enforceable provisions of Directives 2014/53 / EU (April 16, 2014), 2011/65 / EU, of the European Parliament and the Council, transposed to the Spanish legislation by Royal Decree 188/2016, of May 6, 2016. To see the complete declaration of conformity, you can access it through the following link:

http://www.innovacelular.com/images/web/descargas/descargas/declaracion/declaracion_swr4m.pdf

Guarantee letter

GUARANTEE LETTER

DEVICE
ITEM: _____
Serial number: _____

ESTABLISHMENT
Purchase date: _____

91 499 06 24

INNOVA CELULAR SL
C/ Severo Ochoa Nº 9 nave 8B
28521 Rivas-Vaciamadrid
Madrid

Establishment signature